



CONTOUR[®]PLUS ONE 智能血糖儀與 CONTOUR[™]DIABETES app 應用程式配對



© 版權所有 2018 Ascensia Diabetes Care Holdings AG。

www.contourplusone.hk

Blood Glucose Monitoring System

Contour

DIUS NNF»

ŝ * ••••00 Ξ My Readings + Today, Saturday, Oct 21 5.2 mmol ľ 11:25 My Week (Past 7 Days) ÷ \checkmark ♠ My Averages

每當Bluetooth[®] (藍芽)連接可用時,您的血糖儀 與 CONTOUR DIABETES app 應用程式同步

Contour

plus n

Blood Glucose Monitoring System

◀)

配對您的血糖儀

1.

在配對您的CONTOUR PLUS ONE 血糖儀前 請開啟您的流動裝置上的Bluetooth[®] (藍芽)

前往"設定","藍芽",按下切換按鈕 開啟Bluetooth[®] (藍芽)



Contour plus one Blood Glucose Monitoring System

配對您的血糖儀

1.

在配對您的CONTOUR PLUS ONE 血糖儀前 請開啟您的流動裝置上的Bluetooth®(藍芽)

前往"設定","藍芽",按下切換按鈕 開啟Bluetooth[®] (藍芽)





配對您的血糖儀

1.

在配對您的CONTOUR PLUS ONE 血糖儀前 請開啟您的流動裝置上的Bluetooth®(藍芽)

前往"設定","藍芽",按下切換按鈕 開啟Bluetooth[®] (藍芽)

	Θ •	
•0000	11:25	© 1 🗖
	Settings	
\rightarrow	Airplane Mode	\bigcirc
Ŷ	Wi-Fi	Off >
*	Bluetooth	Off >
((_Å))	Mobile Data	>
ଡ	Personal Hotspot	Off >
S.	Carrier	>
	Notifications	>
	Control Center	>
C	Do Not Disturb	>



配對您的血糖儀

1.

在配對您的CONTOUR PLUS ONE 血糖儀前 請開啟您的流動裝置上的Bluetooth®(藍芽)

前往"設定","藍芽",按下切換按鈕 開啟Bluetooth[®] (藍芽)

•0000	11:25	@ 1 🕞
	Settings	
	Airplane Mode	\bigcirc
Ŷ	Wi-Fi	
*	Bluetooth	Off >
(⁽ Å)	Mobile Data	
0	Personal Hotspot	Off >
	Carrier	>
	Notifications	>
	Control Center	>
C	Do Not Disturb	>
-		



◀)

配對您的血糖儀

1.

在配對您的CONTOUR PLUS ONE 血糖儀前 請開啟您的流動裝置上的Bluetooth[®](藍芽)

前往"設定","藍芽",按下切換按鈕 開啟Bluetooth[®] (藍芽)

•		•	
••000 EE 奈	11:25	* ∎	D
Settings	Bluetooth		
Bluetooth		\bigcirc	I
			/

Contour plus one Blod Glucose Monitoring System

配對您的血糖儀

1.

在配對您的CONTOUR PLUS ONE 血糖儀前 請開啟您的流動裝置上的Bluetooth[®](藍芽)

前往"設定","藍芽",按下切換按鈕 開啟Bluetooth[®] (藍芽)

		-
••••• EE 🔶	11:25	* 📼
< Settings	Bluetooth	
Bluetooth		
	and a second	

Contour plus one Blod Glucose Montoring System

配對您的血糖儀

於**CONTOUR DIABETES app**應用程式,建立帳 戶後,您將被要求配對血糖儀。

當準備配對您的血糖儀時,請先關閉血糖儀並將 其靠近您的流動裝置 。



r	• •	
•••• EE 🗢	11:25	* 💼 +
<		X
	Pair a Meter	Í
O	Ć	
Make keep	sure your meter is off it near your mobile de	and vice.
	Continue	





配對您的血糖儀 選取"繼續"







www.contourplusone.hk

 \checkmark

配對您的血糖儀

長按CONTOUR PLUS ONE 血糖儀上的中間按 鈕,直到看到試紙接口出現閃爍藍燈才放開。







配對您的血糖儀

3.

長按CONTOUR PLUS ONE 血糖儀上的中間按 鈕,直到看到試紙接口出現閃爍藍燈才放開。





 \checkmark

配對您的血糖儀

長按CONTOUR PLUS ONE 血糖儀上的中間按 鈕,直到看到試紙接口出現閃爍藍燈才放開。





Contour plus one Blod Glucose Monitoring System



3.

配對您的血糖儀

4.

選取您的血糖儀序列號 。

您可以在血糖儀背面的標籤找到序列號。



Θ	
••∞ EE ≈ 11:25 < Select your Meter	* ••• + ×
Tap the name and serial numb that matches your device.	ber
123 C ontour7802-P040188	~
Continue	

Contour plus one Blod Glucose Monitoring System



4.



0		.]
•••• EE 🗢		* 🕞 +
Sele	ect your M	eter (i)
		ıl number levice.
Contour	7802-P040188	~
	Continue	



© 版權所有 2018 Ascensia Diabetes Care Holdings AG。

5.

配對您的血糖儀

CONTOUR DIABETES app 應用程式現在將連接到血糖儀





Contour plus one Blod Glucose Monitoring System

6. 配對您的血糖儀

Bluetooth[®] (藍芽)配對請求訊息會顯示在屏幕上



	6				
•••		11	:25		* 💼 +
<					
		4	1,		
	Blueto "Contour7	oth Pa 804-P04	iring Ro 0188″ wo	e quest ould like	to
	pa	ir with yo	our iPhor	ne.	
	Cance	el		Pair	

Contour plus one Blod Glucose Monitoring System





	• — •
	Bluetooth Pairing Request "Contour7804-P040188" would like to pair with your iPhone.
	Cancel Pair
l	



© 版權所有 2018 Ascensia Diabetes Care Holdings AG。

6.

記對您的血糖儀 選取"配對"來確定配對請求

6.





Contour, plus plus, Blod Glucose Monitoring System

©版權所有 2018 Ascensia Diabetes Care Holdings AG。

配對您的血糖儀

7.

當您的血糖儀已經成功配對 選取"繼續"

您現在可設定CONTOUR DIABETES app 應用程式。



0 •	
••••• EE 🗢 11:25 👋 🔳	- , +
Meter Paired Successfully	
DO NOT turn your meter off as you continue setup.	
Continue	
·····	



© 版權所有 2018 Ascensia Diabetes Care Holdings AG。

7.

配對您的血糖儀

當您的血糖儀已經成功配對 選取"繼續"

您現在可設定CONTOUR DIABETES app 應用程式。



• — •
••••○ EE 🗢 11:25 🔋 🕞 f
Meter Paired Successfully
Note: A solid blue light shines on your meter for a few seconds.
DO NOT turn your meter off as you continue setup.
Continue



如果您對CONTOUR®PLUS ONE智能血糖儀和 CONTOUR[™]DIABETES app應用程式有任何疑 問,請咨詢您的醫療保健專業人員,並參閱產品 使用指南。 如慾了解更多資訊或查詢您的國家是否有 CONTOUR®PLUS ONE血糖儀,請瀏覽 www.contourplusone.hk



請變更目標範圍,飲食和運動前先諮詢您的醫療保健專業人員

獲取這個程式



Apple 與 Apple 標誌為蘋果公司(Apple Inc.) 所屬商標,並已於美國和其他國家註冊。App Store 為蘋果公司(Apple Inc.) 的服務標記,Bluetooth®之文字標記和標誌為藍芽技術聯盟 (Bluetooth SIG, Inc.) 擁有的註冊商標,健臻醫療保健 (Ascensia Diabetes Care) 對上述標誌的任何使用均須依據授權規定。Google Play 與 Google Play標誌為Google 公司 (Google Inc.) 所得商標。

Ascensia、Ascensia Diabetes Care 標誌以及Contour均為健臻 (Ascensia Diabetes Care Holdings AG) 的商標和 及/或註冊商標。





